

Outdoor

Single rail for straight stairs



Stannah

Make a step change

If a Stannah stairlift could make life easier, why wait?


Life is better when you have the energy to do the things you want to do, like pottering in your garden or entertaining your family and friends outside.

Our experience has taught us that the sooner a stairlift is installed, the sooner you will be able to start enjoying the things you love doing. Just think how a Stannah stairlift could make your day-to-day life easier.

A stairlift that's right for you and your home.

We know that not all homes are the same. If you think your stairs are unsuitable for a stairlift, we think you might be pleasantly surprised by our slimline and stylish designs.

- Our slimline rails and fold-away chairs make the best use of available space and leave enough room for others to use the staircase.
- All our rails are tailor-made just for you, so they fit your staircase perfectly.
- Contrary to popular belief our rails are fitted to the staircase and not the wall - ensuring minimal mess and bother.
- Installation is quick and easy.
- Our outdoor stairlift can safely operate in all seasons.

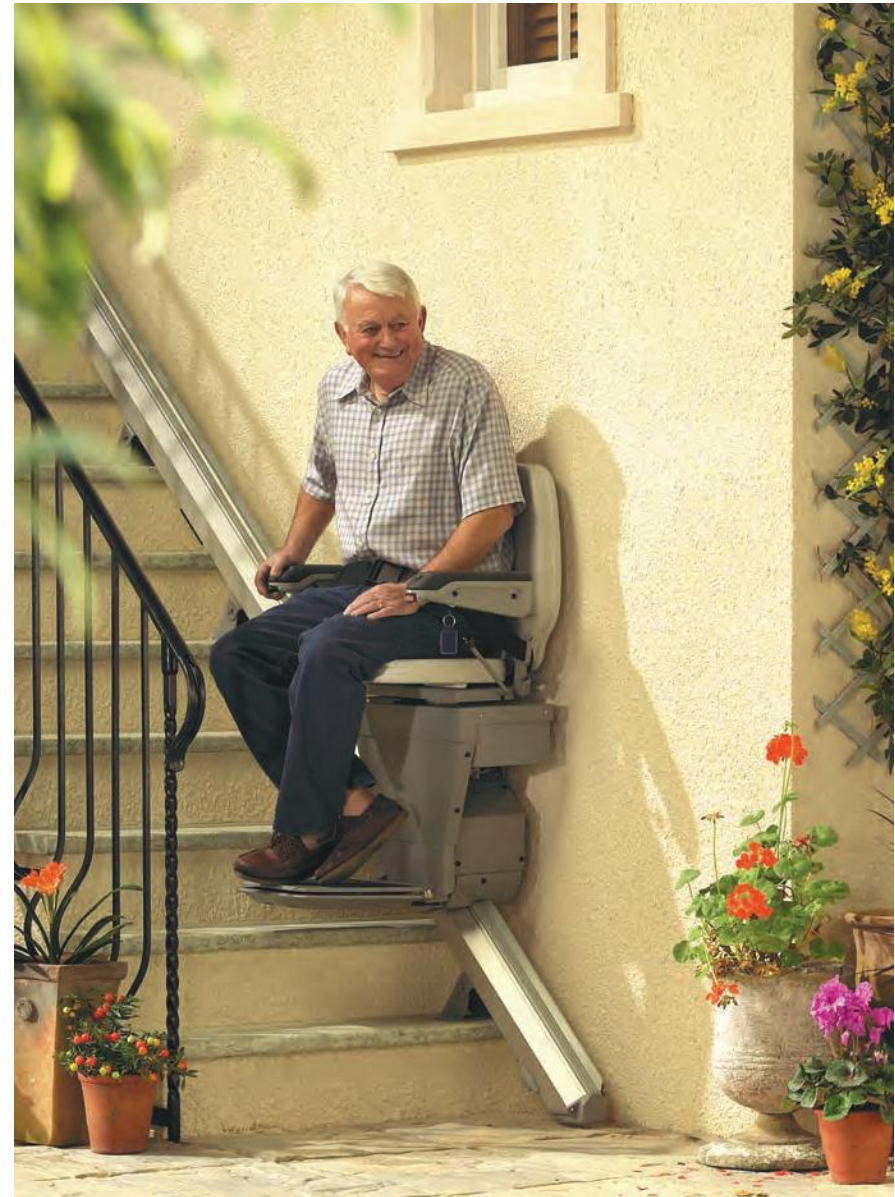

A gentle touch of the controls is all it takes to climb your stairs

A company you can trust

We are a British, family-run and owned company and have been for five generations.

The current managing directors are the great, great grandchildren of Joseph Stannah, who started the company in the 1860s.

His engineering principles and strong family values run as true today as they always have.

Now, over 150 years later, we have manufactured more than 500,000 stairlifts and installed products in over 40 countries.

Outdoor: Robust, reliable and comfortable

Our Outdoor stairlift for straight stairs is specially designed to work all year round. An ideal solution for either a flight of outside stairs or for just a few steps down to your garden.

The swivel seat allows you to get off the chair in complete safety at the top of your stairs. The joystick controls can even be positioned on either arm, whichever is more comfortable for you.

A protective cover is also provided to help you to keep your stairlift clean and dry when you're not using it.

Fitted to the stairs and not to the wall, the Outdoor straight rail can only be fitted to solid concrete, solid wood or metal staircases.



The Outdoor's slimline design allows others to still use the stairs

Outdoor key features

1. Directional controls

Right-handed or left-handed, we can put the joystick controls on either arm, whichever you prefer, just another thoughtful touch.

2. Manual swivel seat

Using the levers enables you to swivel the seat easily, making it simple to get off the chair.

3. Folding seat, footrest and arms

To store the stairlift away, you can fold away the seat, arm and footrest, keeping the stairway clear for other users.

4. Safety edges

If there is something blocking the chair on the stairs, the safety edges recognise this and the stairlift will stop automatically.

5. Keyswitch

The stairlift can be immobilised when you're not using it for additional security.



6. Seatbelt

For complete peace-of-mind, our Outdoor stairlift comes equipped as standard with a safety seatbelt.

7. Call station (optional)

You can choose to have a call station to enable you to call the stairlift for yourself, or send it to someone else.



Outdoor key benefits

Effortless and simple. This sums up beautifully just how much thought and care we have designed into each and every function of our Outdoor stairlift. From the safe and easy-to-use seatbelt, to a stairlift that takes up minimal space on your staircase, we have made your Outdoor stairlift simple, safe and reliable.



On-off light as standard for ease of use



The Outdoor features a weatherproof cover



A removable power key adds additional security

"We are delighted with the stairlift. My wife enjoys the ride. It gives her independence. She finds it very easy to use"

Mr A, Cirencester



The easy way to climb your stairs

Everything about our Outdoor stairlift is made to make managing your stairs as easy, comfortable and safe as it can be, so your life changes for the better the moment you begin to use it.

From the simple controls at your fingertips to reliable operation and a flexible and compact design, we have done all the hard work so you don't have to. Just settle into the comfortable seat and sit back, as you travel up or down your stairs with ease.

We really have taken care of every detail to ensure your Outdoor stairlift is as effortless to use as it is easy to operate.

Step 1: Simply pull down the footrest, arms and seat base.

Step 2: When the seat base and footrest are down, sit on your stairlift and make yourself comfortable in the seat. Then fasten the easy-to-use

centre lapbelt and you're now ready to safely go.

Step 3: A gentle movement of the directional control is all it takes for you to travel effortlessly and safely up or down your stairs.



Step 4: When you have arrived at the top of your stairs, you can then use your swivel lever to rotate you onto your

landing ready to get off your stairlift in a safe position.

Step 5: When you are not using your stairlift, it can remain in position at the top of your stairs to act as a safety barrier, or, it can be folded away to allow space for other stair users.

“Now we have our Stannah stairlift, we no longer have to face an unwanted move into single storey accommodation”

Mr & Mrs H, Worcestershire

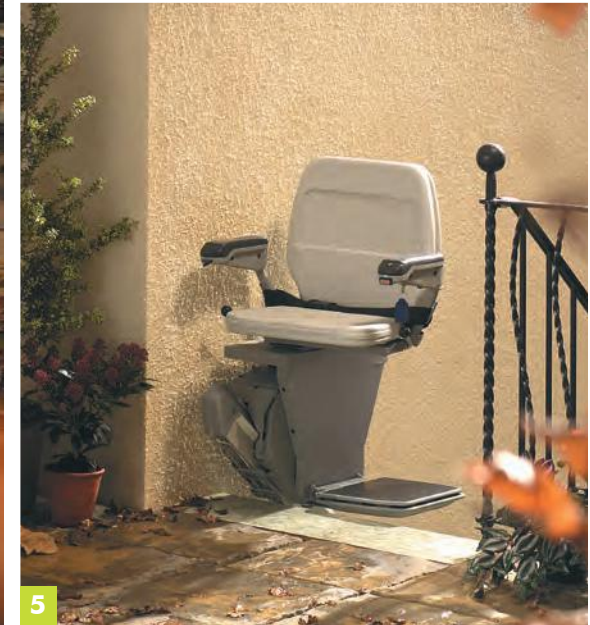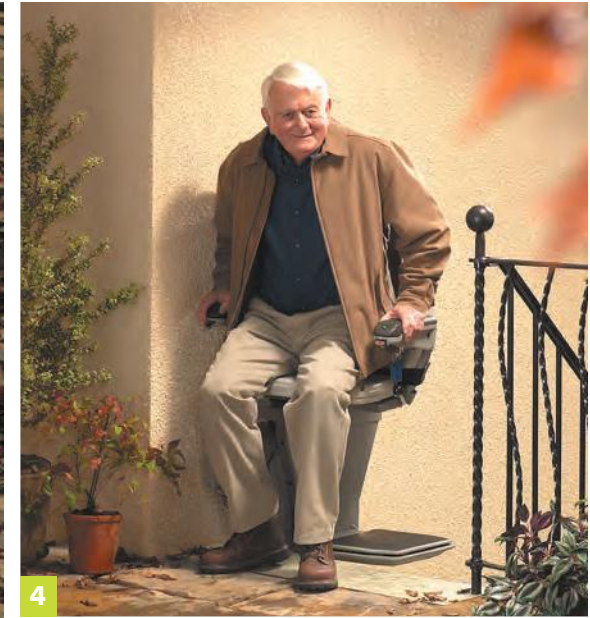



Our company policy is one of continuing product improvement. We therefore reserve the right to alter the specifications and to change and improve design at any time. Full operating instructions are supplied for all products. Stannah is a registered trademark. The CE mark indicates that this product complies with the applicable European directives which relate to health, safety, environment and consumer protection. Design, manufacture and installation: Stannah Stairlifts Ltd has been accredited to ISO9001:2008 since 2009, this is for design, development, production and distribution of stair lifts. Alterations in models, standard and optional equipment as described in the text and illustrations, may occur. For precise information please contact Stannah.

© Stannah Stairlifts Ltd 2020. Not to be reproduced wholly or in part without the written permission of Stannah Stairlifts Ltd Issue 1 / January 2020

